



Clinical **Pocket guide**

English

The background features a light green color with several overlapping rounded rectangular shapes in a slightly darker shade of green. A prominent white path or line starts from the top center, curves down and then right, crossing the green shapes, and continues horizontally across the bottom of the page.

Summary

Introduction	4	Ventilation settings configuration	29
Quick Start	5	• Expiratory Trigger	30
The EO-150 device	6	• Pressurisation Ramp	31
Circuit Configuration	8	• Volume Target Speed	32
Turn ON/OFF the device	10	• Inspiratory Time	33
Access the Clinical Menu	11	Presets Configuration	34
Access the Patient Configuration	13	Monitoring and Waveforms	35
Circuit Calibration	14	Supplemental Oxygen	36
Configure the Settings and Alarms view	15	Advanced Monitoring (FiO ₂ / SpO ₂ / tcCO ₂)	37
Settings and Alarms Configuration	16	Data Download	38
Ventilation modes configuration	17	External battery	39
• ST Mode	18		
• VTS Mode	19		
• PAC Mode	20		
• CPAP Mode	21		
• C-Flow Mode	22		
• (A)VCV and (A)PCV Mode	24		
• PSV Mode	25		
• PSV VT Mode	26		
• V-SIMV and P-SIMV Mode	27		
• MPV and MPP Mode	28		

Introduction

The EO-150 ventilator device provides continuous or intermittent ventilation support for pediatric and adult patients weighing at least 3.5 kg (8 lbs) who require mechanical ventilation. The device is intended to be used in home, institution and hospital, for both invasive and non-invasive ventilation.

The information in the Pocket Guide is not intended to exceed the instructions in the medical prescription and User Manual.

The EO-150 ventilator is considered strictly medical device and is intended to be used by trained and qualified personnel under medical supervision.


If any operating and maintenance problems are identified, the local representative should be contacted.

An alternative source of ventilation should be available for dependent patients. Failure to do this may result in patient injury or death.




Clinicians are advised to read the User Manual (Clinician) before using the EO-150 ventilator.

Quick Start


In case of emergency, the ventilation module can be directly turned on simply by pressing the  button on the module keyboard.



2

To start ventilation
Click on  on the keyboard.

1

Press  on the front panel keyboard to turn on the device.
The home screen displays.

The EO-150 device



- 1** Air inlet and filter
- 2** AC / DC Power connector
- 3** USB port
- 4** O₂ input
- 5** FiO₂/SpO₂ connector
- 6** Remote Alarm connector



- 1** Display screen
- 2** Ventilation module
- 3** Proximal pressure, valve, and proximal flow connectors
- 4** EO device housing unit
- 5** Inspiratory / Circuit Port
- 6** ON / OFF switch
- 7** Power source indicator
- 8** High priority alarm indicator
- 9** Technical alarm indicator
- 10** Circuit alarm indicator
- 11** Ventilation start / stop
- 12** Physiological alarm indicator
- 13** Medium priority alarm indicator
- 14** Alarm reset
- 15** Battery level indicator

The home screen



- 1 Patient Trigger Indicator
- 2 Patient Profile Indicator
- 3 Circuit configuration
- 4 Ventilation Mode
- 5 Alarm signal
- 6 Alarm indicators
- 7 Clinical mode status

- 8 Battery life indicator
- 9 Power source indicator
- 10 Time
- 11 Alarm tab
- 12 Alarm inhibition
- 13 Active settings information
- 14 Preset selector

- 15 Airway pressure indicator
- 16 Help button
- 17 Power Off button
- 18 Time and date
- 19 Event log
- 20 Menu button
- 21 Settings short cut

Circuit Configuration

Connect a patient circuit **without** flow sensor on the exhalation line



Single limb circuit with leak or mouth piece



Single limb circuit with expiratory valve
(without proximal flow sensor)

Use only circuits components compatibles for use with the EO-150.
It is recommended to perform the calibration at each switch of circuit type.

Connect a patient circuit **with** flow sensor on the exhalation line




Single limb circuit with expiratory valve
(with proximal flow sensor)



Double limb circuit (with adapter)


Turn ON the device

Ensure that the device has been charged prior to use or connect to AC power or DC connector inlet.

1. Insert AC connector into power inlet.
2. Turn the screw lock clockwise to secure.
3. Device will turn on automatically. If starting on battery, press  on front panel keyboard to power on the ventilator.




Turn OFF the device using the Touch Screen


1. Press  and hold until the circle becomes red.
 2. Validate.
- The ventilator turns off and the Home Screen turns into sleep mode.



To start ventilation using the Home Screen

1. Press  on the touch screen.
2. Ventilation starts.



To stop ventilation using the Home Screen


1. Press and hold  until:
 - The red line around the Start/Stop key completes a full circle.
2. Validate the displayed stop ventilation confirmation pop-up.
3. Ventilation stops.




Access the Clinical Menu

Unlock the Clinical Menu

1. Click on the Menu icon .
2. Hold down the button  until the red circle is complete.

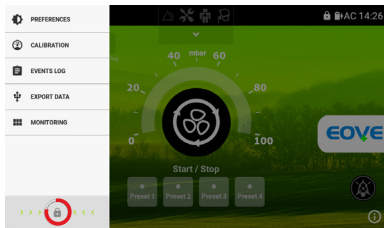


3. Click on Validate.

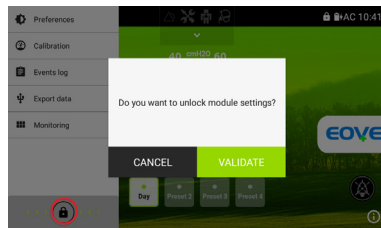



NOTE:

Access to the Clinical Menu is reserved for the responsible clinician.

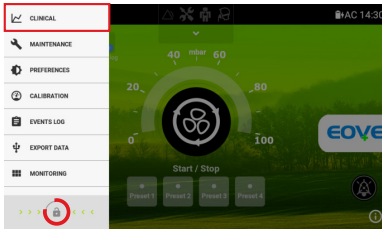
Access the Clinical Menu



The Clinical Menu could be accessible by Menu  or by Home Screen  .
After 15 minutes the Menu will be locked automatically.

Access the Patient Configuration


- 1 Patient profile: Adult or Pediatric
- 2 Newborn Option
- 3 Circuit type: Valve, Leak or Mouth Piece
- 4 Ventilation type: Volume or Pressure
- 5 If the Nominal Leak is configured, the Leak can be set between 0 and 100 LPM or AUTO (40 LPM). Considering IPAP of 15 cmH₂O.*
- 6 Activation of the FiO₂ sensor
- 7 Perform circuit calibration

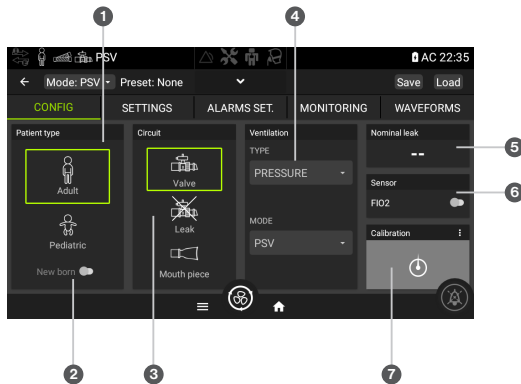


* In the case of NIV, check the Leak in the manual for each mask in advance.

Circuit Calibration

From this screen, you can select the configuration and perform a calibration. Simply press on the calibration button in the grey area.

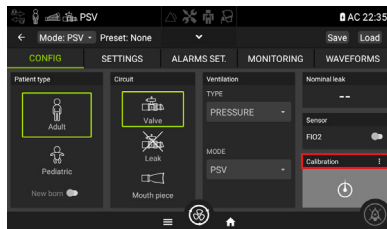

Follow the 2 steps calibration process when prompted and press on “Result” to go back to the home page.

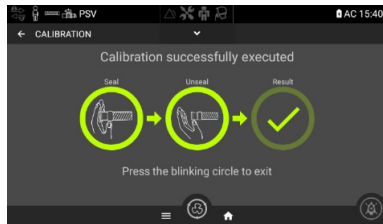

NOTE:

- A calibration must be completed for each new circuit configuration
- Complete the calibration with all circuit components (circuits, filters, humidificator), but without the patient interface.

Configure the Settings and Alarms view


To configure the settings and alarms in list format (settings + monitoring on the same page), access the Preferences menu and activate “Settings in list format”.

1 Settings



Display option
when List is ON



Display option
when List is OFF

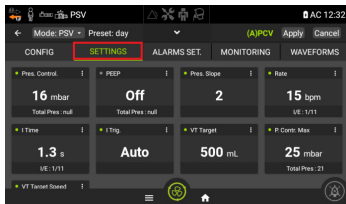

2 Alarms

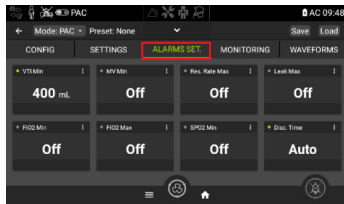
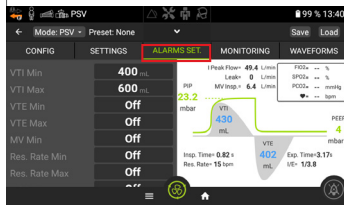


Settings and Alarms Configuration

Settings and Alarms tabs

These tabs allow to set the parameters for the ventilation settings and the alarm settings.

When settings are displayed as a list, the right part of the screen displays the monitored data in real time.

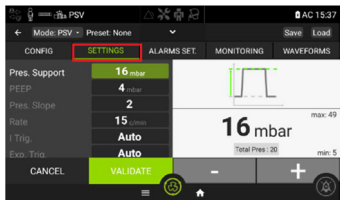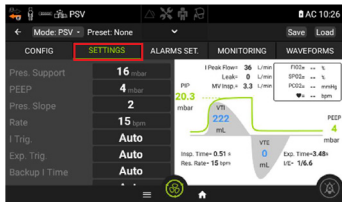

1. From the SETTINGS, click on one of the settings values.
2. Adjust the setting with + and -.
3. Press on VALIDATE to apply the changes.


Some settings could be turned to AUTO or OFF, according to need.

Ventilation modes configuration

Depending on the type of circuit selected, the following ventilation modes are available



S(T)	Spontaneous Timed (with leak)
VTS	Volume Target Spontaneous (with leak)
PAC	Pressure Assisted/Controlled (with leak)
CPAP	Continuous Positive Airway Pressure (with leak)
C-FLOW	Continuous Flow (with leak)
(A)VCV	Volume Assisted Controlled Ventilation (with expiration valve)
(A)PCV	Pressure Assisted/Controlled Ventilation (with expiration valve)
PSV	Pressure Support Ventilation (with expiration valve)
PSV VT	Pressure Support Ventilation Volume regulated (with expiratory valve)
V-SIMV	Volume Synchronized Intermittent Mandatory Ventilation (with expiratory Valve)
P-SIMV	Pressure Synchronized Intermittent Mandatory Ventilation (with expiratory valve)
MPV	Mouthpiece Volume Ventilation (with expiration valve and leak)
MPP	Mouthpiece Pressure Ventilation (with expiration valve and leak)

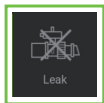

ST Mode: Spontaneous Timed


EPAP = 6 mbar

IPAP = 25 mbar

RR = 10 bpm

1 = Assisted breath triggered and cycled by the patient
2 = Mandatory breath triggered by backup respiratory rate

The modes available in the leakage circuits

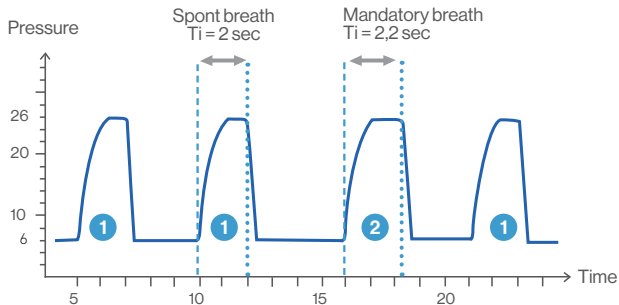





VTS Mode: Volume Target Synchronised

EPAP = 6 mbar
IPAP min = 15 mbar
IPAP max = 25 mbar
Vol. Target = 500 ml
RR = 10 bpm

1 = Pressure increment breath by breath
2 = Targeted volume reached

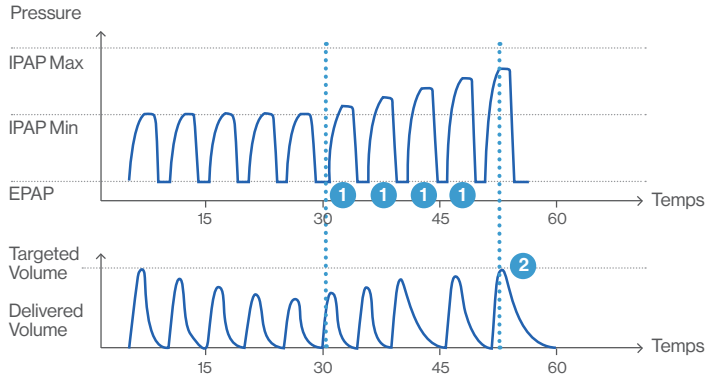

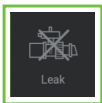

PAC Mode: Pressure Assisted/Controlled


EPAP = 6 mbar


IPAP = 25 mbar

RR = 10 bpm

1 = Assisted breath triggered by patient and cycled by ventilator


2 = Mandatory Breath triggered by respiratory rate

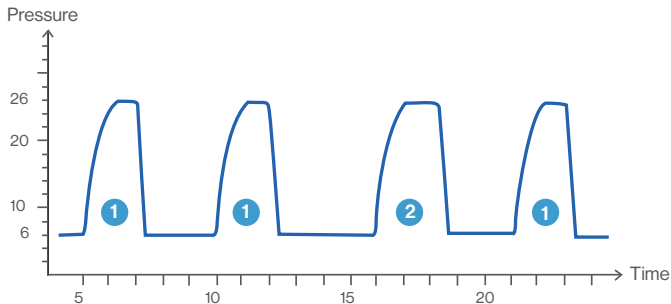


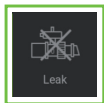

CPAP Mode: Continuous Positive Airway Pressure


In the Continuous Positive Airway Pressure (CPAP) mode, the device delivers a continuous pressure to the patient at all times. All breaths in this mode are spontaneous breaths.

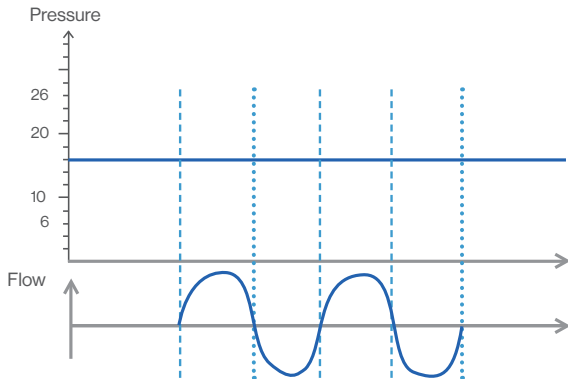

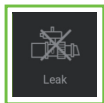

C-Flow Mode: Continuous flow

This mode delivers continuous flow to the patient. The set flow (Flow) is delivered continuously through the humidifier and nasal cannula. All breaths in this mode are spontaneous breaths.

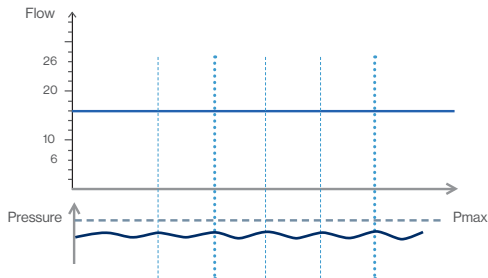


Settings	Flow (l/min)
Adult	10-60
Pediatric	2-25

Gas mixture concentration (in % of FiO₂)


FiO ₂ % theoretical value	C-Flow setting (L/min)							
		10	15	20	30	40	50	60
4	53	42	37	32	29	27	26	
8	84	63	53	42	37	34	32	
12	-	84	68	53	45	40	37	
16	-	-	84	63	53	46	42	
20	-	-	-	74	61	53	47	
25	-	-	-	87	70	61	54	
30	-	-	-	-	80	68	61	
40	-	-	-	-	-	84	74	
50	-	-	-	-	-	-	87	

O₂ concentration is given by :

$$\% \text{O}_2 = \frac{\text{O}_2 \text{ Flow} \times 100 + (\text{Flow setting} - \text{O}_2 \text{ Flow}) \times 21}{\text{Flow setting}}$$

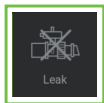

! C-Flow mode needs to be used with a humidifier

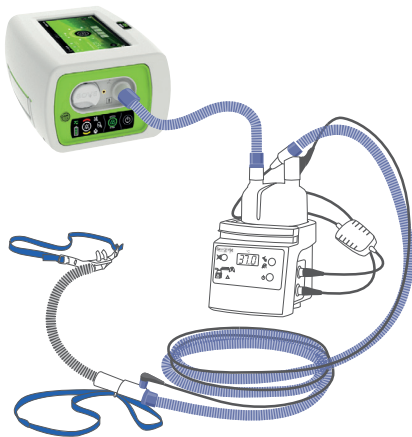


How to set up from the humidifier:

1. Connect the short circuit to the inspiratory port of the ventilator.
2. Connect the heat circuit, temperature probes and electric adapter.
3. Place de nasal cannula.

* Use the leak adapter (cap) included into the starter kit (EO LMP2).



Select **“invasive mode”** on the humidifier (according to the models)

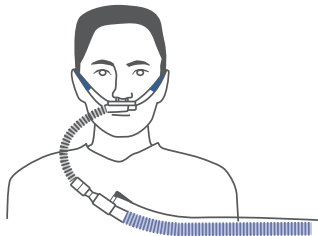




(A)VCV Mode: Volume Assisted/Controlled Ventilation Circuits with an Expiratory Valve

VT = 600 ml
PEEP = OFF
RR = 12 bpm
Ti = 2 sec
Flow Ramp = 1 Square


- 1 = Assisted breath triggered by the patient and cycled by the ventilator
- 2 = Mandatory breath based on respiratory rate

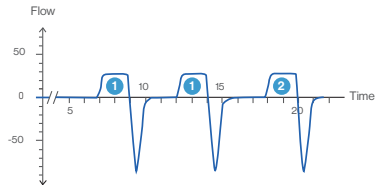

(A)PCV Mode: Pressure Assisted/Controlled Ventilation

Pres. Control. = 20 mbar
PEEP = 6 mbar
Pres. Slope = 2
RR = 10 bpm

- 1 = Assisted breath triggered by the patient and cycled by the ventilator
- 2 = Mandatory breath triggered by respiratory rate

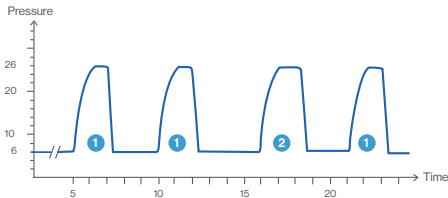



PSV Mode: Pressure Support Ventilation


Pres. Support = 20mbar

PEEP = 6mbar

RR = 10 bpm

1 = Assisted breath triggered and cycled by the patient

2 = Mandatory breath triggered by back up respiratory rate

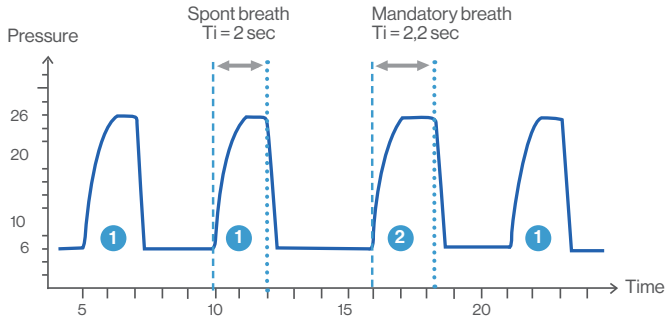



PSV VT Mode: Pressure Support Ventilation Volume Regulated

Vol. Target = 600 ml

Pres. Sup min = 20 mbar


Pres. Sup max = 35 mbar

RR = 10 bpm

PEEP = 6 mbar

1 = Pressure increment breath by breath

2 = Targeted volume reached

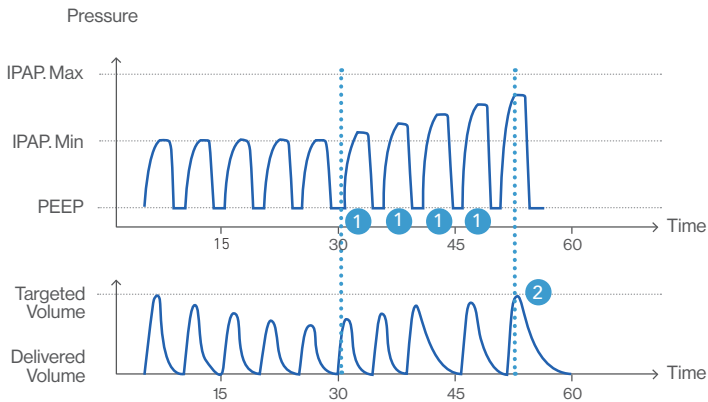




V-SIMV Mode: Volume Synchronized Intermittent Mandatory Ventilation

$VT = 600$ ml
 $PEEP = 6$ mbar
 $RR = 10$ bpm


- 1 = Assisted breath triggered by the patient
- 2 = Mandatory breath based on respiratory rate but synchronized with patient

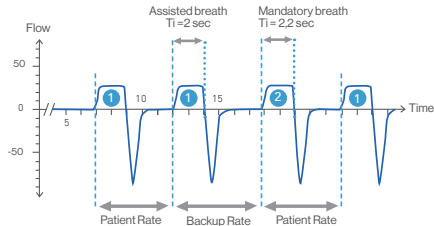

P-SIMV Mode: Pressure Synchronized Intermittent Mandatory Ventilation

$IPAP = 25$ mbar
 $EPAP = 6$ mbar
 $RR = 10$ bpm

- 1 = Assisted breath triggered by the patient
- 2 = Mandatory breath based on respiratory rate but synchronized with patient

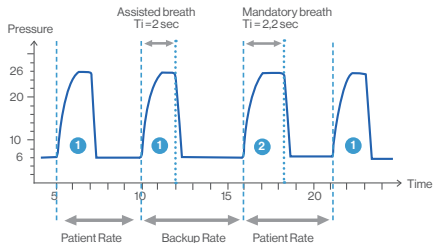




MPV Mode: Mouth Piece Volume Ventilation

VT = 600 ml
RR = OFF
Min.Pres = 5 mbar

1 = Breath triggered by the patient
2 = Disconnection alarm

Setting	Positive Trigger (± 0.2 mb)	Negative Trigger (± 0.2 mb)
Auto	0.4	-0.7
1	0.3	-0.4
2	0.4	-0.7
3	0.6	-1.2
4	0.8	-1.8
5	1.0	-2.4

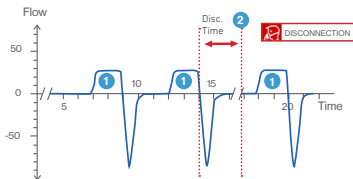

IT'S POSSIBLE TO SWITCH OFF THE DISCONNECTION ALARM AND THE RR.


In MPV and MPP modes, triggers in pressure can be negative or positive, the increase of the bias flow will promote the positive trigger. The bias flow "OFF" setting only promotes the negative trigger.




MPP Mode: Mouth Piece Pressure ventilation

Pres. Control = 25 mbar
RR = OFF
Min.Pres = 5 mbar

1 = Triggered cycle
2 = Disconnection alarm

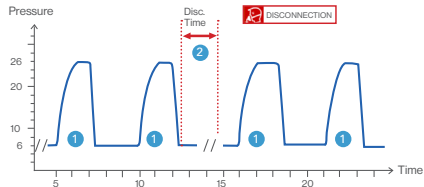

Ventilation settings configuration

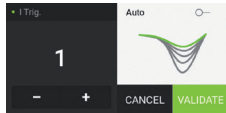


Inspiratory Trigger: Manual setting adjustment

The sensitivity levels can be adjusted from 1 to 5 (1 = Most sensitive – 5 = Less sensitive)

Trigger sensitivity is adapting to the exhalation patient's flow curve for more comfort:

1. Beginning of the exhalation: Inhibition time based on previous inspiration characteristics trigger to safely detect early triggering.
2. Optimal trigger: Dynamic sensitivity optimization adapting to the patient's waveform to allow breath triggering in optimal conditions.

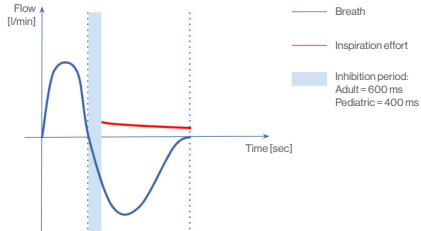


Trigger setting	1	2	AUTO	3	4	5
Sensitivity threshold	1,2-0,5	1,3-0,6	1,4-0,7	1,4-0,7	1,5-0,8	1,6-0,9

Adult minimum inhibition time = 600 ms

Pediatric minimum inhibition time = 400 ms


Expiratory Trigger: Manual setting adjustment


Cycling threshold to expiration is based on % of peak flow. When the inspiratory flow decreases and reaches a percentage of the peak flow (green line), expiration phase is activated.

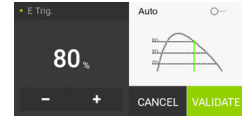

Expiratory Trigger: Auto setting

In Auto mode the cycling threshold is computed according to 2 parameters: Peak Flow and given T_i max. When flow decreases and cross the line between T_0 and T_i max, expiration phase is activated.




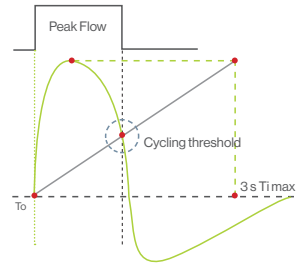


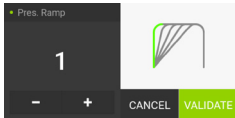

Rise time: Manual setting adjustment

Pressurisation Ramp levels can be adjusted from 1 to 5
(1 = Fastest- 5 = Slowest)




	Setting	1	2	3	4	5
Adult	Duration (ms)	100	200	300	400	500
Pediatric	Duration (ms)	50	100	150	200	250

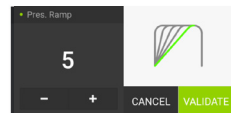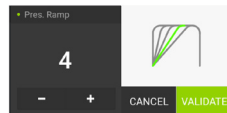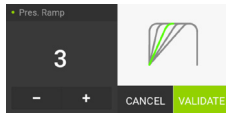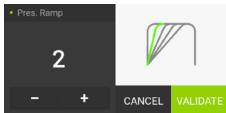

Volume Target Speed: Manual setting adjustment

This parameter sets the pressure increment to reach the Target Volume.
The sensitivity levels can be adjusted from 1 to 3 (1 = Slowest – 3 = Fastest)
Two increments are implemented from each setting:

Setting	Volume > 80% of Target Volume	Volume < 80% of Target Volume
1	0.5 mbar	0.5 mbar
2	0.5 mbar	1 mbar
3	0.5 mbar	2 mbar

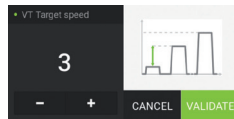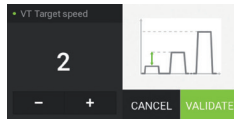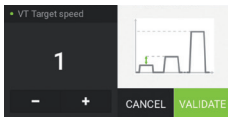

Inspiratory Time: Manual and AUTO setting adjustment

The inspiratory time can be adjusted from 0.3 to 2.5 seconds.


In **AUTO** mode, minimum inspiratory time is set at:

I. Time Min = Pressure ramp time +100 ms

Maximum inspiratory time is set at:

I. Time Max = $\frac{30}{\text{Given respiratory rate}}$


in order to avoid Inspiration/Expiration reversed settings

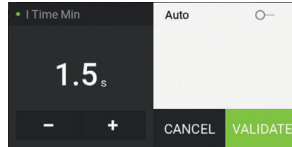

Backup Inspiratory Time: Manual and AUTO setting adjustment

The backup inspiratory time can be adjusted from 0.3 to 2.5 seconds.

In **AUTO** mode, auto Expiratory Trigger is applied.

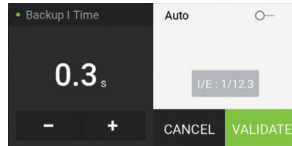


Presets Configuration

It is possible to configure different Presets by selecting different types of circuit, ventilation modes and alarm settings, which could be altered during ventilation.




From the upper band of the clinical menu, you can:

- 1 Save the active mode as preset and rename the saved preset.
- 2 Load a previously saved preset (to visualize the settings contained in the preset).
- 3 Delete the applied preset.

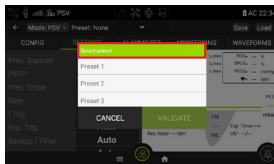

Select a preset

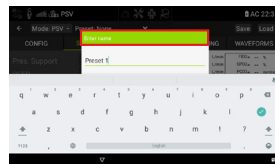


Rename a preset


Monitoring and Waveforms

- 1 Flow in red
- 2 Pressure in blue
- 3 Curve selection: pressure, flow and volume
- 4 Switch between waveforms and loopgraph
- 5 Pause/Play
- 6 Zoom reset

Monitoring

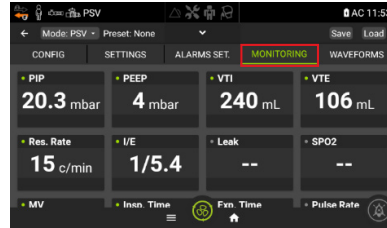

Waveforms

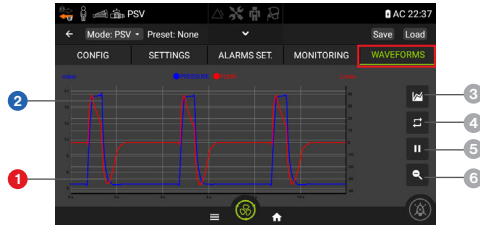

A zoom can be done by using pinch-to-zoom (touch two fingers on the surface, and while continuing to touch the surface, move them apart to zoom in or together to zoom out.)

Supplemental Oxygen

To add supplemental oxygen:

1. Unlock the oxygen input at the rear of the device by pushing up on the locking clip.
2. Plug in the oxygen adaptor (supplied with the EO 150) to the oxygen input.
3. Attach the end of the oxygen supply tube (provided with the EO 150) to the oxygen adaptor.
4. Attach the other end of the oxygen supply tube to the oxygen source.
5. Start ventilation.
6. Turn on the oxygen and adjust for the prescribed flow rate or FiO_2 level.

To remove supplemental oxygen:

1. Turn off the oxygen source.
2. Unlock the low flow oxygen input at the rear of the device by pushing up on the locking clip.
3. Remove the oxygen adaptor from the oxygen input.



ATTENTION

- Use only medical grade oxygen.
- Ventilator must be started and stopped after oxygen supply has been turned off.
- Oxygen can be added up to a maximum flow of 20 l/min.

Advanced Monitoring (FiO₂ / SpO₂ / tcCO₂)

One of the advantages of the EO-150 is that it enables advanced ventilation monitoring. It is possible to attach Optional Accessories for monitoring FiO₂, Pulse Oximetry and Transcutaneous Capnography.

Monitoring FiO₂

ATTENTION

The optional FiO₂ sensor should be used to ensure the concentration of oxygen delivered to the patient, providing alarms for maximum and minimum oxygen concentrations.

To connect the FiO₂ sensor:

1. Plug the FiO₂ cable into the FiO₂ port.
2. Plug the FiO₂ sensor into the other end of the FiO₂ cable.
3. Attach the T-adaptor to the Inspiratory Patient Port.
4. Plug the FiO₂ sensor into the T-adaptor.

NOTE:

- In order to display the FiO₂ measurements and to set the alarms, activate FiO₂ monitoring in the Patient/Circuit configuration menu. (See page 13)
- A 5 minutes delay after the beginning of ventilation is recommended to ensure that FiO₂/SpO₂ recording is correct.
- Use the FiO₂ Kit Cable + FiO₂ cell + Cell T Piece Adaptor.

Monitoring Pulse Oximetry (SpO₂)

ATTENTION

Only use compatible NONIN finger pulse sensors.

To connect the pulse oximeter:

1. Connect the Nonin finger sensor selected on the Nonin XPOD cable.
2. Connect the plug of the Nonin XPOD cable to the SpO₂ connector at the rear of the device.
3. Connect the sensor to the selected patient finger.

Monitoring CO₂ Transcutaneous

ATTENTION

For the monitoring of tcCO₂ use the Sentec PCO₂ monitor and accessories.

To connect the Sentec PCO₂ monitor is necessary:

1. RS323 / USB converter cable (for more technical information check the User's Manual).
2. tcCO₂ sensor according to patient profile: Only use compatible SENTEC sensors.

These are optional accessories, check the user manual for more information.

Data Download





USB stick Specs:

- FAT32
- 32 GB max
- Class 10 max
- 1 partition

1. Insert a USB key in the USB port at the rear of EO-150
2. In the Menu tab, select «EXPORT DATA»
3. Create a new file if required
4. Select one or more files
5. Export the selected files
6. Import the data for EOVE Clinical Software


NOTE:

- The EO-150 saves a data file each day. It's possible to manually save a new file by pushing on the "New file" button.
- Each file contains the last 24 hours of detailed waveforms data and up to one year of trends.
- The EO-150 memory is able to store up to 32 files.
- Selecting "Export USB" will automatically transfer all data stored on the USB key. You can select a single file or multiple files.
- For access to the EO-150 Clinical Software, please contact your local representative.
- For Telemonitoring, please contact your local representative to check the availability for each region.

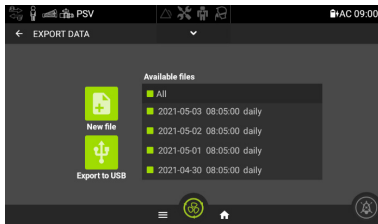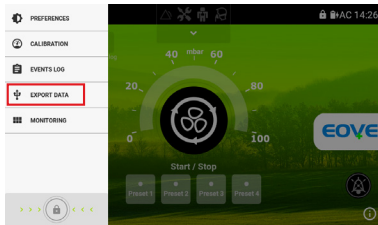

External Battery

For extra mobility, the EO-150 ventilator provides external batteries as an optional accessory.

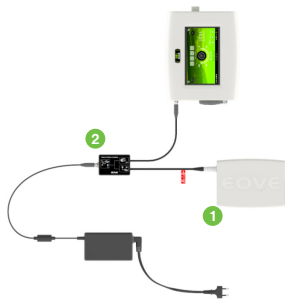

- MATERIAL NEEDED:**
- 1 EO-150 BATTERY PACK (EO-BATPCK - include External Battery (EO-BAT9), Battery Cable, Power Source Cable)
 - 2 BATTERY Y CABLE (EO-EXTBATCBL)
 - 3 EO-150 TRAVEL BAG (EO-TRVELBAG1X0-EVO)
 - 4 BATTERY BAG (EO-BATBAG)



The BATTERY Y CABLE (EO-EXTBATCBL) could be used to recharge both the device and the external battery.




Travel bag



Battery bag

The BATTERY PACK COUPLING CABLE (EO-CPLPACK) could be used instead of the BATTERY Y CABLE (EO-EXTBATCBL), losing the possibility to recharge the device and the battery. It is possible to connect 2 External Batteries simultaneously if necessary.

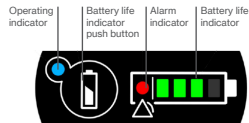

Keyboard view

EXTERNAL BATTERY (EO-BAT9) rear view

Manufactured by

EOVE
Helioparc
4, Boulevard Lucien Favre
64000 Pau
France
www.eove.fr



EO-150 provides continuous or intermittent ventilation support for adults and children weighing at least 3.5 kg needing mechanical ventilation, at home, in a facility or hospital, and in portable environments, for invasive or non-invasive ventilation.

Medical device class IIb - CE 0459 - Manufactured by EOVE. Please read the user manual.