

# Quick Start Guide



This guide does not  
replace the User Manual

Intended for all the people  
authorized to adjust settings

English

## Turn ON the device



Press on



## Turn OFF the device



Press and hold



+ VALIDATE

## Manage the ventilation



To start ventilation: press on




To stop ventilation: press and hold




+ VALIDATE.

## Unlock the Clinical Menu

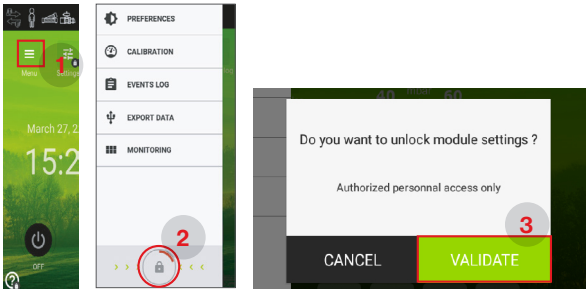


Press on the **Menu** button (1), then **unlock Clinical Menu** pressing and holding (2) and **VALIDATE** (3).

## Setup a New Patient or Circuit Type

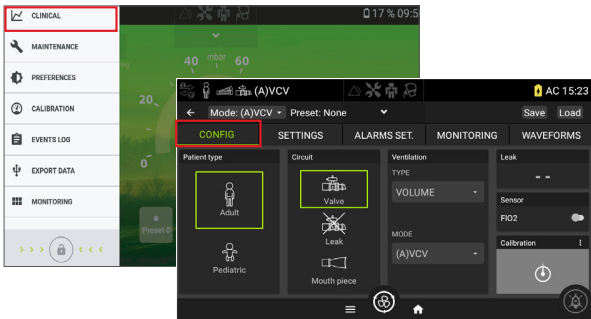

Select **CLINICAL**, then **CONFIG** and set the configuration of Patient Circuit Type and the Ventilation Mode.


## Perform Calibration / Autotest

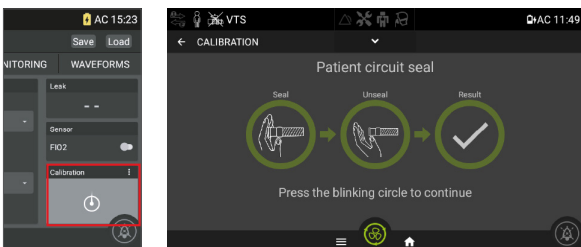

Follow the steps displayed on the screen in order to perform a calibration, then press on **Result**.


The calibration shall be done after every circuit configuration change.

## Setup the Ventilation Parameters

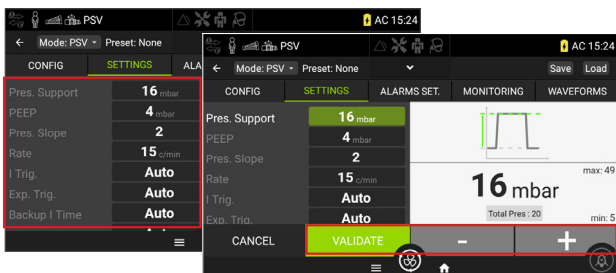


From the tab **SETTINGS**: choose the **ventilation settings**. Select the parameter to be set, adjust with + or -, then press on **VALIDATE** in order to validate the setting.

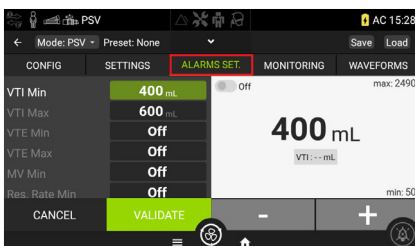

## Setup the Alarms



From the tab **ALARMS SET.**, set the alarms, then press on **VALIDATE**.

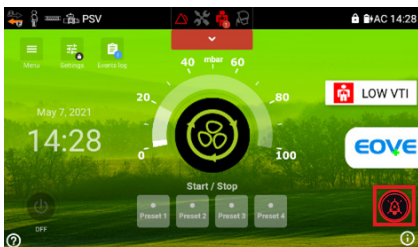
## Silence the Alarms



**Inhibition of the active alarm:** press on   
**Preventive inhibition (2 min):** press and hold  + **VALIDATE**

## Circuit configurations

**Note:** The EO-150 allows the use of every circuit that successfully completed calibration.

 ..... **Single limb circuit with leak or mouth piece**




 ..... **Single limb circuit with expiratory valve (without proximal flow sensor)**



 ..... **Single limb circuit with expiratory valve (with proximal flow sensor)**




 ..... **Double limb circuit (with adapter)**



The double limb adapter is single-patient use. In case of use on several patients, please use an antibacterial filter on the expiratory limb in order to protect the adapter.


## Disable Auto Lock of the Clinical Menu

The automatic lock of the Clinical menu (patient mode activation after 15 minutes of inactivity) can be deactivated from the **Maintenance** menu of the device, with the access code “1586”.

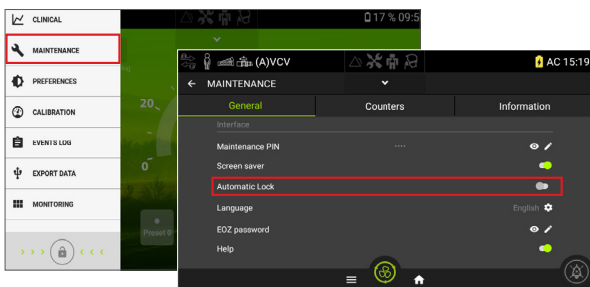

## Configure the settings and alarms view

To configure the settings and alarms in list format (settings + monitoring on the same page), access the Preferences menu and activate “Settings in list format”.

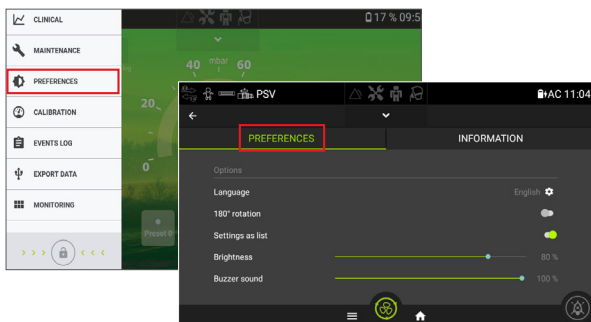

### Manufactured by

EOVE  
Helioparc  
4, Boulevard Lucien Favre  
64000 Pau  
France  
[www.eove.fr](http://www.eove.fr)



EO-150 provides continuous or intermittent ventilation support for adults and children weighing at least 3.5 kg needing mechanical ventilation, at home, in a facility or hospital, and in portable environments, for invasive or non-invasive ventilation.

Medical device class IIb - CE 0459 - Manufactured by EOVE. Please read the user manual.